

Steriblef®

Is a micellar cleansing foam used for the mechanical removal of ocular secretions from the eyelids and eyelashes. Steriblef® can be used in cases of Blepharitis or as part of a daily routine for the care and hygiene of eyelids.

Steriblef® contains natural ingredients ;

- **Tea Tree Oil** to reduce the risk of bacterial infection
- **Cornflower** acts as a gentle anti-inflammatory agent
- **Allantoin** which has soothing, regenerative and healing properties
- **Vitamin E** which is an antioxidant and lipophilic
- **Vitamin B5** helps to repair cells and tissues

Steriblef® is a registered medical device and is presented in a 50ml pump bottle.



How to use

Wash hands carefully before use and clean fingertips with Steriblef.

Press pump and put the foam on clean fingers and gently massage eyelids and eyelashes. Keep the eye closed during application. Leave foam for 60 to 80 seconds before rinsing off with warm water.

Repeat on the other eye

Repeat use 2 or 3 times a day for 7 days

Do not use in cases of hypersensitivity to any of the ingredients

Do not use after 6 months of opening